

## Leselupe

Die Leselupe ist eine Lesehilfe für Menschen mit Sehbehinderung.

### Einschalten der Leselupe

Legen sie das Medium auf die Lesefläche und schalten sie die Lupe mit Druck auf (1) ein.

### Einstellen von Kontrast, Helligkeit und Farbkontrast

Um den Kontrast des Bildes zu erhöhen oder zu verringern, drücken sie die Pfeiltaste rechts oben (2).

Um die Helligkeit zu verändern, drücken sie die Pfeiltasten bei (4).

Für den Farbkontrast drücken sie die Pfeiltasten bei (5).

### Vergrößern und Verkleinern

Um das Bild zu vergrößern oder zu verkleinern, drücken Sie die Tasten + / - bei (3).

## Video Magnifier

The Video Magnifier is a reading aid for people with visual impairment.

### Turn on the reading laser

Place the media on the reading surface and turn on the video magnifier by pressing (1).

### Adjust contrast, brightness and color contrast

To increase or decrease the contrast of the image, press the arrow button on the top right (2).

To change the brightness, press the arrow buttons at (4).

For color contrast, press the arrow buttons at (5).

### Zoom

To zoom the image in or out, press + / - at (3).



### Haben Sie noch Fragen?

Bitte wenden Sie sich an die Ausleihe der Teilbibliothek 1.

### Do you have further questions?

Please refer to the personel at the Help Desk of the Branch Library 1.